Prospecto del Kit de Uso Personal

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


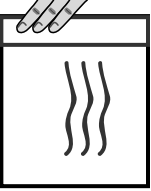
- Package Insert for Personal Use Kit
- Read all package insert directions carefully before sample collection.
Test results may be invalid if test is not performed properly.
- Sample Deposit
- 1. Place supplied collection paper inside toilet bowl on top of water.
 - 2. Deposit stool sample on top of collection paper.
 - 3. Collect sample from stool before paper sinks and stool sample touches water.
 - 4. Flush. Collection paper is biodegradable and will not harm septic systems.
- Sample Collection
1. Label sampling bottle with name, date of birth, and collection date.

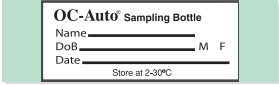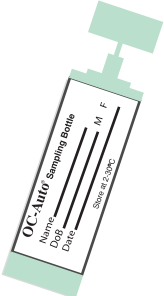
2. Open green cap by twisting and lifting.

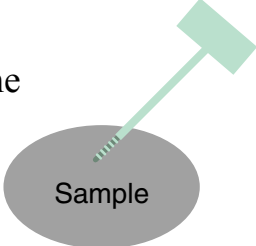3. Scrape the surface of the fecal sample with the sample probe.

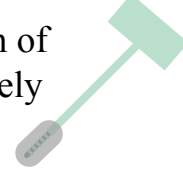
4. Cover the grooved portion of the sample probe completely with stool sample.

5. Close sampling bottle by inserting the sample probe and snap green cap on tightly. Do not reopen.

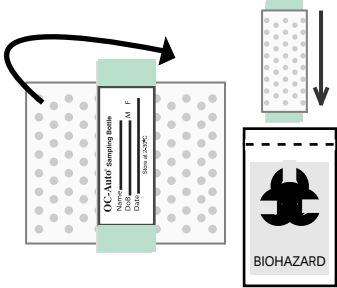
6. Wrap sampling bottle in absorbent pad and insert into plastic bag. Proceed to Mailing Instructions below.


Mailing Instructions

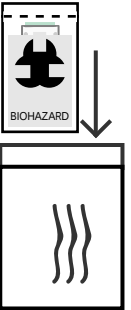
1. Record stool collection date on the test requisition and insert into the mailing envelope, together with the plastic bag containing the sampling bottle and absorbent pad.

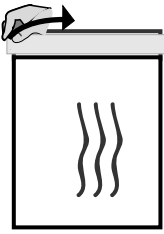
2. Peel tape from flap.

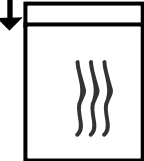3. Fold flap at prefold line.

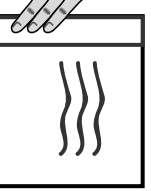4. Press firmly to seal. Promptly (within 24 hours of collection) place in the US Mail for delivery directly to the laboratory.

PN 80609-00