

## **Laboratory Services Important Test Announcement**

## **HEPATITIS A VIRUS ANTIBODIES, TOTAL**

New Test Code: XHAVAB Live Date: 6/18/2024

Effective June 18, 2024, Hepatitis A IgG Antibody, Serum (XHAVIGG) will be deactivated as it is no longer offered by our reference laboratory. Hepatitis A Virus Antibodies, Total (XHAVAB) will replace this test. The Hepatitis A Virus Antibodies, Total detects both IgG and IgM antibodies but does not differentiate between them. XHAVIGG will no longer be orderable.

- Methodology: Qualitative Chemiluminescent Immunoassay
- Performed: Daily
- Turnaround Time: 2 days
- Specimen Required:
  - Collect
    - Whole blood
      - Gold tube
      - Red tube
      - Purple tube (EDTA)
  - Specimen Volume
    - 3 mL
  - Specimen Preparation
    - Centrifuge within 2 hours, remove plasma/serum
  - o Storage/Transport/Temperature
    - Refrigerate
  - Unacceptable Conditions
    - Heat treated specimen, gross hemolysis, gross lipemia, citrate specimens, specimens containing particulate material
  - Stability:
    - Whole blood
      - Room Temperature 2 hours
    - Serum/Plasma
      - Refrigerated 7 days
      - Frozen Indefinitely
  - o Comments:
    - Assay detects both IgG and IgM antibodies but does not differentiate between them.

If you have any additional questions about **XHAVAB**, please refer to the Laboratory Test Directory or call Client Services at 614-722-5477.