




## Laboratory Tube Type Collection Guide

Container	Information	Container	Information	Container	Information
	<p>Tests: CMP, BMP, Hepatic, TSH, proBNP, PCT, BhCG, Glucose and MORE ...</p> <p><b>Label: SST GEL</b></p> <p>Collect: BD Vacutainer SST</p>		<p>Tests: PT, APTT, D-Dimer, Fibrinogen and MORE ...</p> <p><b>Label: Blu Na C</b></p> <p>Collect: BD Vacutainer Buff. Na Citrate</p>		<p>Tests: CBC, Retic Count, ESR and MORE ...</p> <p><b>Label: Lav EDTA</b></p> <p>Collect: BD Vacutainer K2E 7.2mg</p>
	<p>Tests: Zinc, Copper</p> <p><b>Label: DBlu Lav</b></p> <p>Collect: BD Vacutainer Trace Element K2 EDTA 10.8mg</p>		<p>Tests: Selenium</p> <p><b>Label: DBlu Red</b></p> <p>Collect: BD Vacutainer Trace Element Serum</p>		<p>Tests: Type and Screen, ABO Confirmation and MORE ...</p> <p><b>Label: Pink KED</b></p> <p>Collect: BD Vacutainer KD EDTA 10.8mg</p>
	<p>Tests: Latic Acid</p> <p><b>Label: Gry NaFl</b></p> <p>Collect: BD Vacutainer Sodium Fluoride</p>		<p>Tests: Ammonia, Blood Gas (Venous)</p> <p><b>Label: Grn Li H</b></p> <p>Collect: BD Vacutainer Lithium Heparin (no gel)</p>		<p>Tests: Troponin-I</p> <p><b>Label: Grn Li H</b></p> <p>Collect: BD Vacutainer PST Gel and Lithium Heparin</p>
	<p>Tests: Keppra, Lacosamide, and MORE ....</p> <p><b>Label: Red No A</b></p> <p>Collect: BD Vacutainer Serum</p>		<p>Tests: Bili, CMP, and MORE ...</p> <p><b>Label: Pd SST GL</b></p> <p>Collect: BD Microtainer SST Amber</p>		<p>Tests: CBC, Type &amp; Screen and MORE ...</p> <p><b>Label: Pd Pink KEDTA or Lav EDTA</b></p> <p>Collect: BD Microtainer MAP K2E</p>

**Notes:**

- This does not encompass all tubes/containers and tests that are applicable.
- Full Test requirements can be viewed on our test menu website: [www.testmenu.com/nypwestchester](http://www.testmenu.com/nypwestchester)

For any additional questions, please call the laboratory at 914-787-3276