


Aptima® Multitest Swab Specimen Collection Procedure

During Sample Collection:

Be mindful of the following steps for test accuracy

ACCEPT

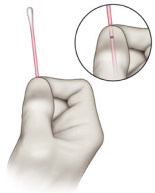

✓ Hold swab, placing thumb and forefinger in the middle of shaft covering score line.



✓ Place swab in the transport tube so the score line is at the top of the tube.



✓ Align score line with top edge of tube and carefully break shaft.



✓ Discard top portion of shaft.



✓ Tightly screw cap onto tube.



✓ Provide a specimen source for unique identification for specimen location.


REJECT



✗ Do NOT pour out tube contents.



✗ Do NOT hold shaft below the score line.

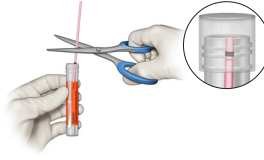

✗ Do NOT cut the swab.



✗ Do NOT break above the score line.




✗ Do NOT fold swab in half.



✗ Do NOT puncture the top.

Bar Code Label Placement

The bar code label must be placed correctly. This allows for accurate sample identification.

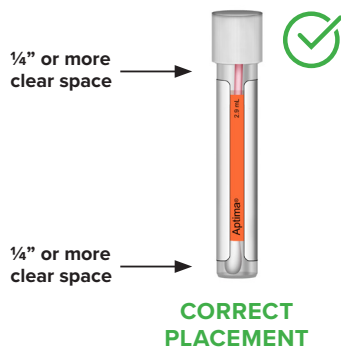

hologic.com | diagnostic.solutions@hologic.com | 1.888.484.4747

Reference: Aptima Multitest Swab. US package insert AW-26253-001. Hologic, Inc.; 2021.

DS-09999-001 Rev. 003 © 2023 Hologic, Inc. All rights reserved. Hologic, Aptima and associated logos are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries. This information is intended for medical professionals in the U.S. and is not intended as a product solicitation or promotion where such activities are prohibited. Because Hologic materials are distributed through websites, eBroadcasts and tradeshows, it is not always possible to control where such materials appear. For specific information on what products are available for sale in a particular country, please contact your local Hologic representative or write to diagnostic.solutions@hologic.com.